sens-aid[®]
by ZiboCare[®]

- EN** Instructions for use - 2
 - LV** Lietošanas instrukcija - 8
 - LT** Naudojimo instrukcijos - 14
 - EE** Kasutusjuhend - 20
-

EN - Instructions for use

Important safety instructions and contraindications

 Always read all the instructions before use

WARNING!

This product may only be used for its intended purpose as described in these instructions. The product must not be combined, assembled or repaired with other parts or accessories than those described in these instructions or in other documentation from ZiboCare.

The product must not be used near fire sources or hot surfaces, such as a fire, burning cigarette or hot lamp. The product may be damaged if it comes in contact with fire sources or heat; you should not smoke while using Sens-Aid.

Always check that the product is intact before use.

Avoid sharp objects that can cause damage to the Sens-Aid.

We do not recommend using Sens-Aid when lying down.

We do not recommend using Sens-Aid while standing or walking, as it has been developed for use in a sitting position, so it is not suitable for use in other ways than in a chair, sofa, wheelchair, armchair or other types of chair.

Check Sens-Aid regularly to make sure it is not damaged. If it is damaged, do not use it until the damage has been repaired.

People with intellectual disabilities and users who cannot communicate their experiences with Sens-Aid may only use Sens-Aid under supervision.

Sens-Aid is not suitable for adults or children who are not able to remove Sens-Aid without difficulty.

1. Introduction

1.1 General information

This product is a medical device and is CE marked according to EU Medical Device Regulation MDR 2017-745. According to the authorities' regulations regarding medical devices, it is the duty of the manufacturer and the health personnel to report accidents and incidents that involve their products. Therefore, we request that all information about any accidents and adverse events relating to our products are reported to ZiboCare immediately.

1.2 Intended use

Sens-Aid is a weight blanket that can be used to advantage at home, at school or in institutions when there is a need for the stimulation that Sens-Aid provides, primarily to create calm - both motor and mental.

Sens-Aid has six heavy wings that wrap around your body and can be arranged to suit your needs, comfort and pleasure.

When using Sens-Aid, you experience a reduction in restlessness, racing thoughts, anxiety and insecurity, and typically experience a widespread feeling of balance and pleasure. The weight of the wings hugs, embraces and soothes the body.

This happens because the hormone oxytocin is released and sends signals to the brain that you can calmly "gear down", find peace and recharge.

Sens-Aid is used in a chair, such as an armchair, school chair or wheelchair, during transport or on a sofa – outside, as well as inside

1.3 Typical applications

Sens-Aid is intended for children and adults of all ages with general reduced well-being, burnout or reduced energy. The user may have a diagnosis, but Sens-Aid is also for people with different challenges, even without a diagnosis. Sens-Aid is often used for:

- Motor restlessness
- Mental imbalance
- Eating disorders
- Concentration problems
- Neurological disorders
- Dementia conditions
- Brain damage
- Mental disorders, e.g., restlessness and anxiety
- Neuropsychiatric diagnoses, e.g., ADHD, ADD, Asperger's Syndrome, Autism and Tourette's Syndrome
- Spasticity
- Racing thoughts
- Involuntary movements
- Social anxiety

1.4 Additional information

When packed in the transport bag, Sens-Aid does not take up much space and is easy to take with you.

Sens-Aid is available in the following version:

Size and weight	Item no.
Adult, 6 kg	4237401JB

2. Features and positions

Once you are sitting down in the seat of your Sens-Aid, you can place the wings over your body to suit your needs, comfort and pleasure. You can either do it yourself or have someone else help you.

What to do:

Placement: Place the Sens-Aid folded in the chair, with the anti-slip back surface facing down before sitting down. Spread the wings over the chair. The upper wings may be folded over the back of the chair, the middle and lower wings folded out over the sides of the chair and any armrests.

Sit in the seat of the Sens-Aid.

Place the lower wings over your thighs. The pockets can be used if you need to rest your arms and hands.

Wrap the middle wings around your stomach. Here, you can also use the pockets for your hands and arms and feel the calm when your hands are in the pockets and you “embrace your body”.

Place the upper wings over your shoulders so that you feel the weight on your shoulders and chest. This will make your breathing deeper, and calm both your body and mind.

ACCESSORIES

Sens-Aid comes in a practical and smart bag with a shoulder strap that makes it easy to take Sens-Aid with you to where you need it most. At the same time, the bag protects the Sens-Aid from dirt and grime.

How to easily fold Sens-Aid to fit in the included bag:

3. Unpacking Sens-Aid

Sens-Aid comes wrapped in recyclable plastic.

1. Start by checking that the packaging is undamaged.
2. Remove the plastic bag and any protective material. Avoid using a knife when opening the packaging, as this may damage the Sens-Aid and bag.
3. Try folding the Sens-Aid out on a chair, and follow the instructions in point 2.
4. Be sure to save this user manual, as well as the cleaning information.

4. Cleaning

It is important to follow the instructions below when cleaning the product; for example, before using Sens-Aid with a new user.

Sens-Aid cannot be washed, but with light superficial dirt and grime it can be wiped off with a well wrung-out cloth. For heavier dirt and grime, Sens-Aid can be dry cleaned.

- Always check before dry cleaning that the Sens-Aid is intact and that there is no damage or tears in the fabric or seams.
- If there is a tear in the fabric or seams, the Sens-Aid cannot be cleaned.

5. Miscellaneous

5.1 Storage

Keep Sens-Aid in the transport bag, in a chair or folded in a place where no one risks tripping over it. Sens-Aid should always be stored indoors in a dry place.

5.2 Labelling

Sens-Aid is labelled according to current EU guidelines in the field of Medical Devices with the product name, production date, item number, size/weight and CE mark. Instructions for use and washing are enclosed.

5.3 Warranty

Sens-Aid comes with a 2-year warranty covering manufacturing defects. The warranty does not cover normal wear and tear, deterioration of the product or damage resulting from careless handling or care of the product.

5.4 Service and maintenance

Sens-Aid requires no service or maintenance other than cleaning as described in these instructions.

5.5 Repair

If the surface of the Sens-Aid is damaged and torn so that the filling comes out, the device can no longer be used. Contact the supplier for more information about a possible repair.

5.6 Recycling

Sens-Aid's fabric covers are fully recyclable and should be recycled as "combustible waste".

5.7 Returns and complaints

In the event of a return or complaint, please contact the shop where the Sens-Aid was purchased. Here, they will help with the further handling of the case according to applicable terms and conditions.

5.8 Symbol explanation

Warning, see important safety instructions

Manufacturer

The production date as indicated on the label

Lot no.

Medical device

See the instructions

The product is CE marked
CE - Class 1, MDR 2017/745

Do not wash

Do not bleach

Do not iron

Do not tumble dry

Dry clean only

Do not heat , e.g., in a microwave

Reference number

TECHNICAL SPECIFICATIONS

Weight:	6 kg
Filling:	70% plastic beads / 30% Polyester
Cover:	100% polyester Oeko-Tex, Standard 100
Standards:	EU Medical Device Regulation MDR 2017-745

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ZiboCare A/S