# relax

by ZiboCare®

- EN** Instructions for use - 2
  - LV** Lietošanas instrukcija - 10
  - LT** Naudojimo instrukcijos - 18
  - EE** Kasutusjuhend - 26
-

## EN - Instructions

# FIND CALM AND BALANCE WITH THE RELAX COLLAR

Congratulations with your Relax sensory collar.

Relax is a newly developed quality product from ZiboCare. It has been developed in a beautiful design and material with a strong background in our many years of experience in the healthcare sector and sensory-stimulating aids, and in close collaboration with users, healthcare professionals and designers.

ZiboCare's sensory products have been developed on the basis of internationally recognised theories of sensory integration and are well documented in research, documentation, testing and cases.

Always read all the instructions before use.

# IMPORTANT SAFETY INSTRUCTIONS AND CONTRAINDICATIONS

Please read all the instructions before use.

The Relax collar may only be used for its intended purpose, as described in these instructions. The product must not be combined, assembled or repaired with other parts or accessories, other than those described in these instructions or in other documentation from ZiboCare.

Always check that the product is intact before use. If there is a tear in the fabric, the collar must not be used or cleaned.

The Relax collar may only be used under supervision for users with poor balance, intellectual disabilities or users who cannot communicate their experiences with the Relax collar. Pay attention to the user's reaction to the collar.

The Relax collar must not be used near fire sources/hot surfaces such as a fire or burning cigarettes. Smoking is not recommended when using the Relax collar.

The Relax collar must not be used to lie down and sleep with. Do not place the collar over the neck when lying down.

The Relax collar must not be used on children under 3 years of age, as it contains small particles.

The Relax collar must not be heated in a microwave or another heat source.

Be careful not to trip over the Relax collar if it is lying on the floor.

Be aware that, before cleaning, on contact with heat, sweat and moisture, the colour may bleed, and slight smudging may occur on light coloured surfaces.

# HOW THE RELAX COLLAR WORKS

## **Purpose and use**

The Relax collar by ZiboCare can be used to help create peace and security for the user. The pressure of the collar helps to give the user a better body sensation and increased well-being.

The basis of all ZiboCare's sensory products, including the Relax collar, is the internationally recognised theory of sensory integration. This theory was developed in the 1970s by A. Jean Ayers, occupational therapist and psychologist from the USA, and is based on how the brain processes sensory impressions and how to influence the various sensory stimuli via weight and pressure in the sense of touch in the skin (the tactile sense) and the sense of movement in muscles and joints (the proprioceptive sense).

Through its evenly distributed weight and non-uniform surface, the collar affects these senses.

## **Application environment**

The Relax collar can be used anywhere - at home, school, after-school centres, institutions, daycare, hospitals and living rooms.

The collar can be used to advantage in teaching situations where the child or young person needs to increase their concentration and focus on an assignment or, for example, at the time of day when energy levels are at their lowest.

The Relax collar can be used in living rooms and/or dining rooms at after-school centres, in institutions, care homes, etc., when there is a need for collection, calm and increased body sensation.

At home, the Relax collar can, for instance, be used for homework, where there is a need to concentrate on the assignments, in eating situations or when there is a need to gear down - relax your shoulders and let the calm wash over you after a hard day at school or work.

## WHO DOES THE RELAX COLLAR HELP?

The Relax collar typically helps users with challenges in the following areas:

Neuropsychiatric diagnoses

- Physical and mental restlessness and/or anxiety
- Dementia conditions
- Physical and mental impairment after a brain injury
- Neurological disorders
- Motor restlessness
- Tension headache
- Concentration problems

The Relax collar can be used regardless of age (but not for children under 3 years) and by people with or without diagnoses.

# PRODUCT VARIATIONS

Size and weight	Item no.
Adult, 1,5 kg	4237400JB

## USE OF THE RELAX COLLAR



The Relax collar is placed over the shoulders, around the neck, with the opening forward on the chest.

The collar can be pulled more or less up the neck and around the throat, depending on individual wishes and needs.

The collar can be put on and worn either in a sitting position or in an upright position. The weight and balance of the collar ensures that it is stable - even during light activity.

If in doubt about the use of the Relax collar or whether the collar will be suitable for a particular situation or challenge, please call ZiboCare customer service, where trained health professionals will provide professional advice and guidance.

### Examples of how to use the Relax collar:

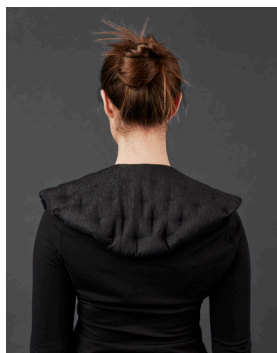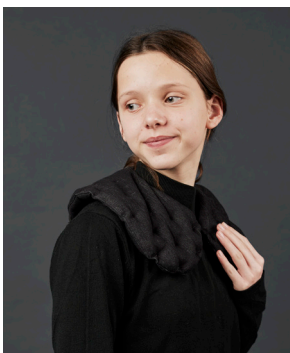# ACCESSORIES

The Relax collar comes in a smart, practical and functional bag, which makes it easy to take the collar with you everywhere - at school and to study, at work, on weekends, on holiday, etc. At the same time, the bag protects the collar from dirt and grime.

## CLEANING AND MAINTENANCE

### Cleaning

The Relax collar is dry clean only. The need for cleaning depends on use and application. If it is passed to another user, it is recommended that the collar should be cleaned.

NOTE! Colour bleeding may occur, which decreases with time, as the colour changes and gains a patina after repeated cleaning.

- Always check that the Relax collar is intact before cleaning and that there is no damage or tears in the fabric or seams.
- If there is a tear in the fabric or seams, the collar cannot be cleaned.

### Maintenance

It is recommended to air the collar regularly.

Apart from as described above, the Relax collar requires no cleaning or maintenance.

# GENERAL INFORMATION

The user and/or patient should report any serious incident that has occurred in connection with the device to the manufacturer and the competent authority of the Member State where the user and/or patient is located.

## **Storage**

To best protect the Relax collar, it is recommended that it be stored in the included bag, which protects against dirt and grime. The Relax collar must always be stored dry. Make sure that the Relax collar does not lie on the floor where there is a risk of tripping over it.

## **Labelling**

The Relax collar is labelled in accordance with EU legislation for medical devices class 1, including, among others, product name, production date, item number, size/weight and CE marking. The label with relevant information, including washing instructions, is sewn into the collar so that the information is always easily accessible.

## **Service life**

The product has an expected service life of 2 years based on correct use, handling, washing and storage as described in the instructions. The service life can be shortened by frequent washing. If there is a tear in the fabric or seams, the collar should not be reused. It is the user's responsibility to continuously assess whether the product can be used beyond the stated service life.

## **Recycling and disposal**

We think about the environment and work based on the UN's Global Goals. In the event of disposal, the Relax collar can be divided into the following individual parts that can be recycled

- Fabric
- Plastic beads

Waste is sorted and disposed of according to local rules

## **Warranty**

There is a 2-year warranty for production defects.

The warranty is void in the following cases:

- Lack of or incorrect maintenance of the Relax collar

- Violent or excessive use of the Relax collar
- Overloading the Relax collar

### Traceability

The label on the product shows the production month and year, as well as the serial number and barcode.

### Returns and complaints

In the event of a return or complaint, please contact the shop where the Relax collar was purchased. Here, they will help with further handling of the case according to applicable terms and conditions.

### Symbol explanation

	Warning, see important safety instructions		Do not wash
	Manufacturer		Do not bleach
	Date of manufacture as stated on the collar's label		Do not iron
	Lot no.		Do not tumble dry
	Medical device		The product can withstand chemical cleaning
	See the instructions		Do not heat, including in a microwave
	The product is CE marked CE - Class 1, MDR 2017/745		Reference number
			Sensory stimulant

## TECHNICAL SPECIFICATIONS

Size: Adult

Weight: 1.5 kg

Filling: Plastic beads

Textile: 46% cotton / 26% viscose / 26% polyester / 2% elastane

Classification: Medical Device class 1

Standards: Complies with the MDR regulation: EU Medical Device Regulation MDR 2017-745

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ZiboCare A/S